

How to use Bonarc™

[Indication for use and effects]

Bonarc® is used for bone regeneration and includes the following indications:

1. Bone regeneration for alveolar cleft and cyst
2. Bone regeneration for the premise of the dental implant placement

1 Opening

After checking for damage, open the aluminum bag and place the case on the clean field.

2 Immersion

Bonarc™ can be used after being immersed into either the patient's blood or saline, taking into account the operability at the time of implantation.


3 Implantation

Refer to the following reference quantities for individual cases.

Sinus floor elevation



(1) Window formation + Schneiderian membrane ablation

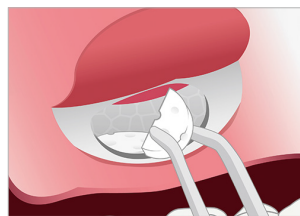

(2) Implant rods or discs



(3) Wound closure



(4) Bone regeneration

Socket preservation



(1) Extraction socket



(2) Implant rods or discs




(3) Wound closure




(4) Bone regeneration

Alveolar cleft

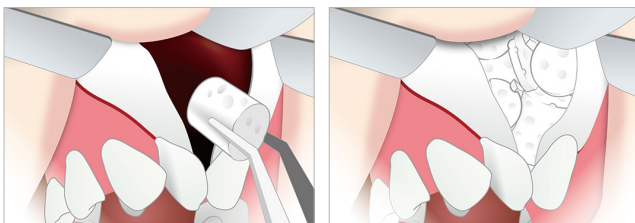

Implant rods or discs in the alveolar cleft

Cyst

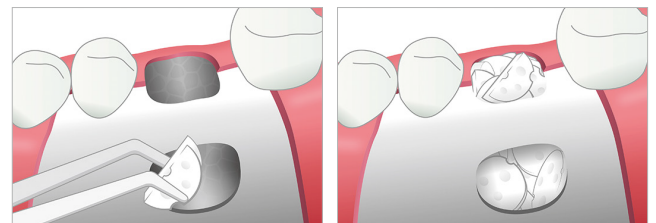

Implant rods or discs into bone defects after cyst harvest